


Set loading guide for the Alaris® Pump module

Alaris® Products

Alaris® Pump module




1 Prime set. Close roller clamp.

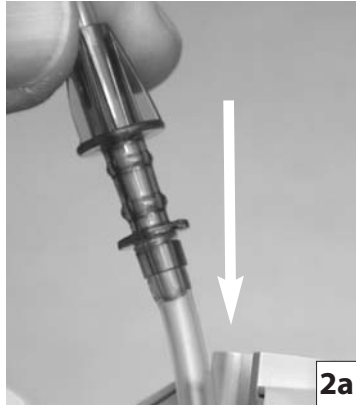

2a Insert set upper fitment vertically into the receptacle at the top of the pump.



3 Ensure fitment is entirely in channel.

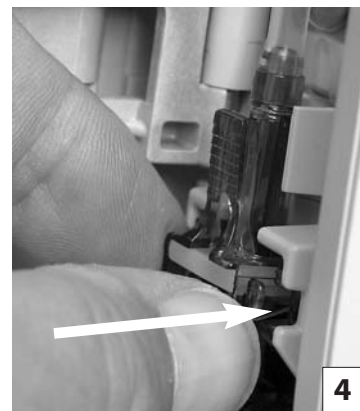

4 Place the safety clamp into receptacle.




5 Press tubing into air-in-line sensor.




6 Do not touch set tubing.



7 Close door and depress latch.

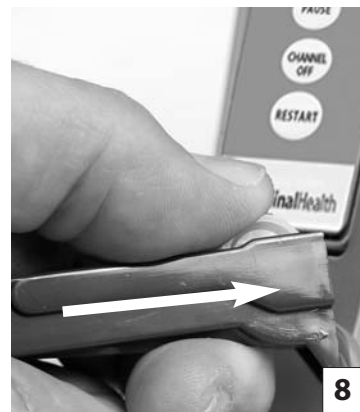


8 Open roller clamp.

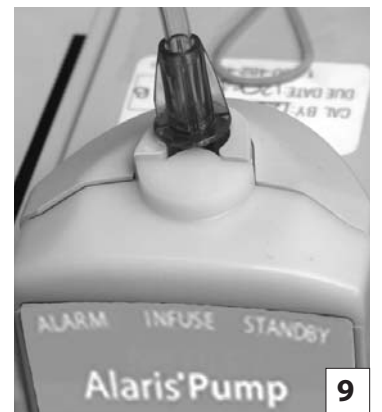

9 Properly loaded upper fitment.

Examples of improper loading or foreign object in door.

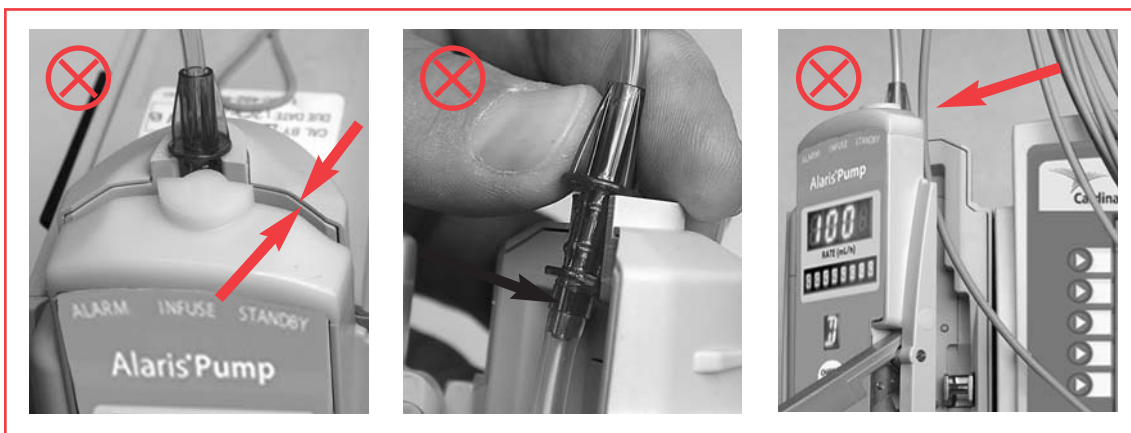

Note that if door is not permitted to fully close unregulated flow or overinfusion may result.

For more information, please contact Customer Advocacy at 800.854.7128, ext. 7812. In Canada at 800.387.8309. Or visit our website at www.cardinalhealth.com.

©2004-2005 Cardinal Health, Inc. or one of its subsidiaries. All rights reserved. Alaris® is a registered trademark of Cardinal Health, Inc. or one of its subsidiaries. 3PCP0168-2 (1105/2M)

♻️ Printed on recycled paper with 10% post-consumer fiber.

Cardinal Health
Alaris® Products
10221 Wateridge Circle
San Diego, California 92121

www.cardinalhealth.com/alaris