



## Alaris® PCA and Alaris® Syringe Modules\*

### Syringe loading tip sheet

#### Step 1

##### Open syringe barrel clamp (clear piece)

- Pull clamp out and hold
- Rotate clockwise or counter clockwise and gently release

#### Step 2

##### Raise drive head (gray in color) to fully extended position

- Twist **gripper control** clockwise and hold
- Raise **drive head** to full extension
- Gently release **gripper control**

#### Step 3

##### Load the syringe

- Insert syringe barrel flange between **barrel flange grippers**

#### Step 4

##### Lock syringe in place

- Pull **syringe barrel clamp** (clear piece) out, hold & rotate until it lines up with syringe
- Gently release clamp against syringe


#### Step 5

##### Lower Drive Head and lock plunger in place



- Twist **gripper control** clockwise and hold
- Lower **drive head** until it makes contact with plunger flange
- Gently release **gripper control**

**Note:** Incorrect selection of syringe manufacturer and size may cause early syringe empty alarm with volume remaining in the syringe.

If the installed syringe is loaded correctly but is not recognized, servicing is required.

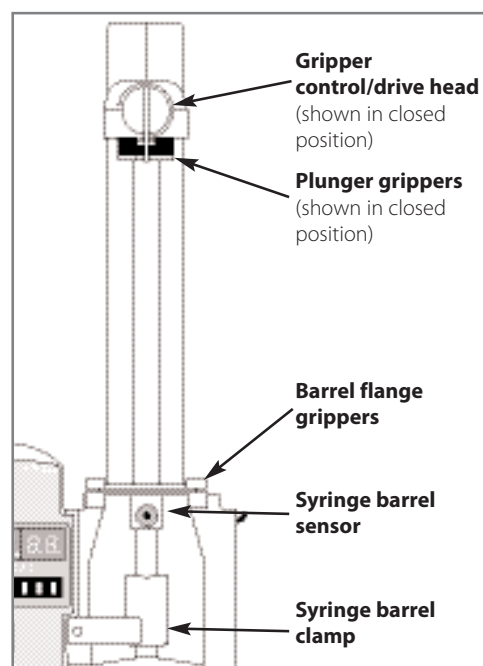

Insert syringe barrel flange between barrel flange grippers

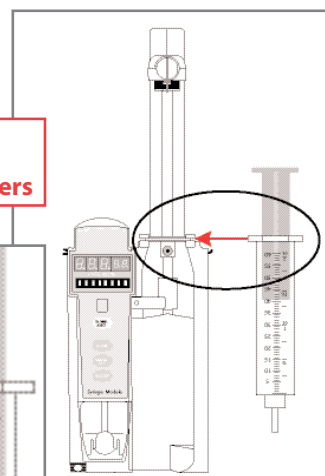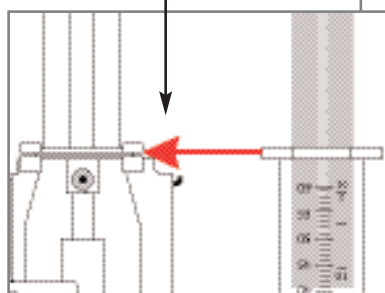

For more information please contact Technical Support at,

U.S.: (800) 854-7128, ext. 6003

Canada: Eastern: (800) 908-9918

Western: (800) 908-9919

\* Alaris® PCA and Syringe Modules formerly known as Medley™ PCA and Syringe Modules.

©2004 Cardinal Health, Inc. or one of its subsidiaries. All rights reserved.  
SSM 2201 (1104/2M)

♻️ Printed on recycled paper with 10% post-consumer fiber.

Cardinal Health  
Alaris® Products  
10221 Wateridge Circle  
San Diego, California 92121

[www.cardinalhealth.com/alaris](http://www.cardinalhealth.com/alaris)

